
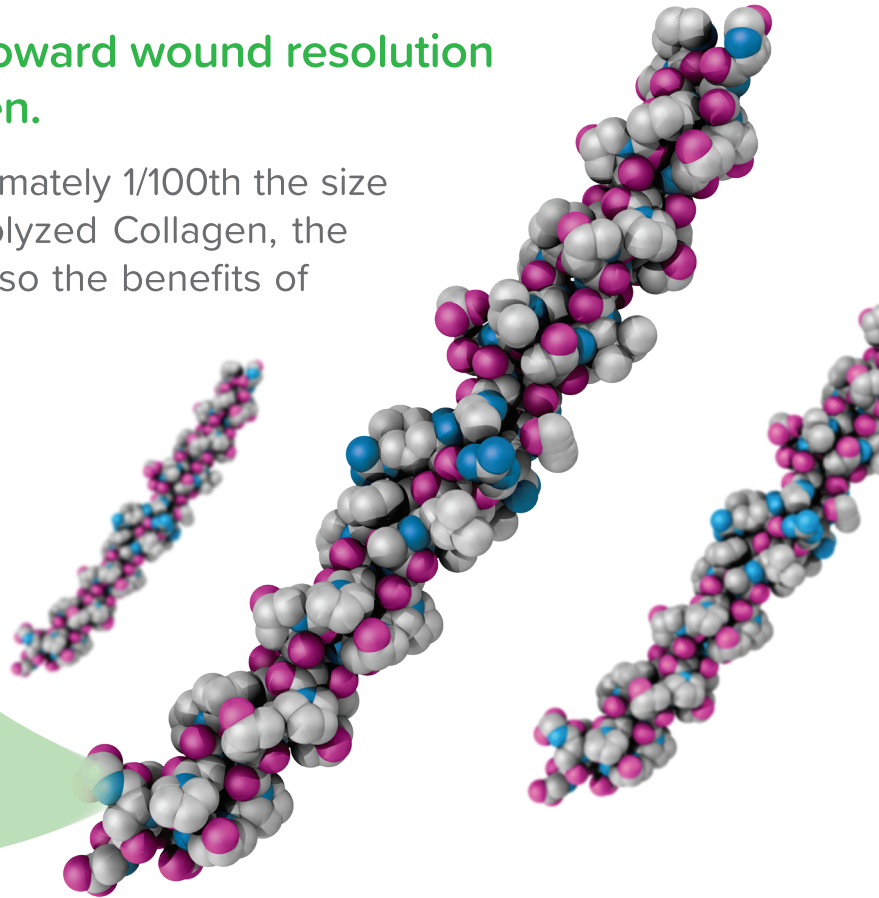


**HYCOL<sup>®</sup>** HYDROLYZED  
COLLAGEN

## THE HYDROLYZED COLLAGEN DIFFERENCE

Give your patients a head start toward wound resolution with **HYCOL<sup>®</sup> Hydrolyzed Collagen**.

HYCOL Hydrolyzed Collagen is approximately 1/100th the size of native collagen. With HYCOL Hydrolyzed Collagen, the body does not have to break it down, so the benefits of collagen are provided to the wound.

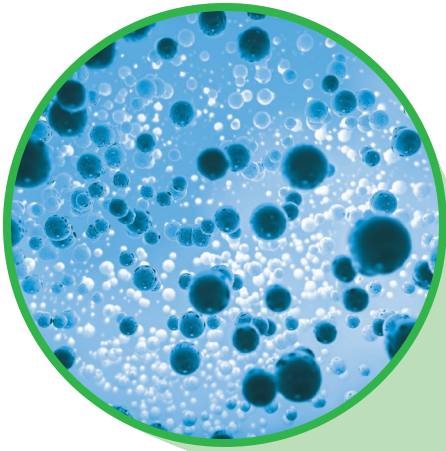

### Safe. Effective. Versatile.


HYCOL Hydrolyzed Collagen is Type I bovine hydrolyzed collagen and contains no additives. It is indicated for:

- Chronic and acute wounds
- Partial- and full-thickness wounds
- Pressure injuries I-IV
- Venous stasis ulcers
- Arterial ulcers
- Diabetic ulcers
- Traumatic wounds
- First- and second-degree burns





## Breaking Down the Benefits

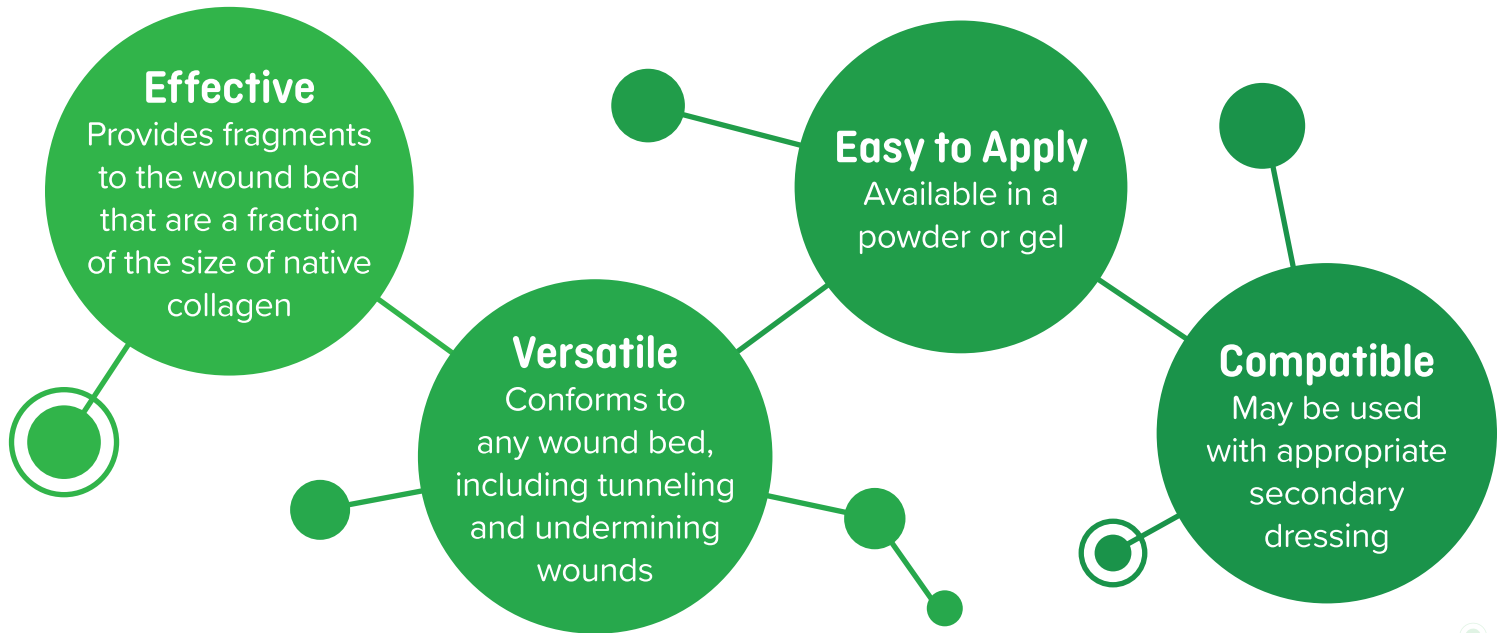


## Application Process



1 Debride the wound bed of slough and necrotic tissue.



2 Apply powder or gel, covering the entire area of the wound bed and wound margin. Gently push powder or gel into any areas of undermining.

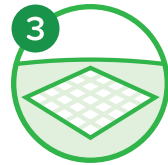

3 Apply an appropriate secondary dressing.



4 Reapply a minimum of 2-3 times a week or as necessary.

## HYCOL Ordering Information

| Product # | Description                                   | Size | Qty/Box | HCPCS |
|-----------|-----------------------------------------------|------|---------|-------|
| HCP0110   | HYCOL <sup>®</sup> Hydrolyzed Collagen Powder | 1 g  | 10      | A6010 |
| HCP0512   | HYCOL <sup>®</sup> Hydrolyzed Collagen Powder | 5 g  | 12      | A6010 |
| HCG1010   | HYCOL <sup>®</sup> Hydrolyzed Collagen Gel    | 10 g | 10      | A6011 |

