



DESCRIPTION

BIASURGE Advanced Surgical Solution is a clear, colorless solution for use with lavage or preferred application method. The product contains PHMB as a preservative to reduce microbial growth within the packaging during shelf storage after the sterile seal has been broken.

INDICATIONS

BIASURGE Advanced Surgical Solution is indicated for use in mechanical cleansing and removal of debris, including microorganisms, from wounds.

CONTRAINDICATIONS

BIASURGE Advanced Surgical Solution should not be used if there is a history of allergy to any of the ingredients.

WARNINGS

DO NOT use for intravenous infusion or injection. DO NOT consume or use in the eyes. BIASURGE Advanced Surgical Solution may only be used for adults and children.

DIRECTIONS

Irrigate surgical site with BIASURGE Advanced Surgical Solution for mechanical cleansing of debris, dirt and foreign materials, including microorganisms; the mechanical action can be provided by either a manual syringe or powered irrigation device. Cumulative use should not exceed 30-days. OPTIONAL WARMING: BIASURGE may be warmed to 40°C (104°F) for up to 30 days before use. Once removed from a temperature warming cabinet, BIASURGE should be used or discarded.

STORAGE

Store at room temperature (approximately 25°C or 77°F). Do not refrigerate or freeze. Discard bags after procedure.

INGREDIENTS

BIASURGE Advanced Surgical Solution contains Purified Water, Poloxamer 407, Sodium Chloride, Ethylhexylglycerin, Hypromellose, Octane-1, 2-diol, Polyaminopropyl Biguanide (Preservative: PHMB: 0.1% w/w), Edetate Disodium (EDTA), Edetate Trisodium (EDTA).

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

GLOSSARY



*Sterile Fluid Path
by Autoclave*



*Prescription
Use Only*



*See Instructions
for Use*



*Single Patient
Multiple Use*



*Store at Room
Temperature*



*Do Not Use if
Damaged or Opened*



*Warning: Not
For Injection*



*Warning: Not
for IV Use*

Distributed by:

Sanara MedTech Inc.
1200 Summit Ave., Suite 414
Fort Worth, TX 76102
1-800-205-7719

BIASURGE is a trademark of Sanara MedTech Inc.
© 2023 All rights reserved. Patent on file.
BSG231107PR